


Boxer's Fracture Application


Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. This device should not be worn by persons with known allergies to neoprene.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

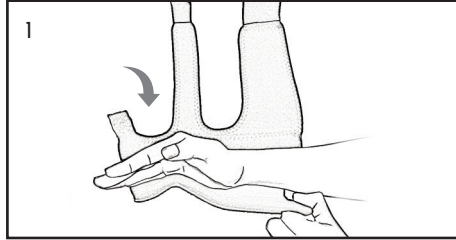


1. Apply the brace, wrapping the shortest strap around the little and ring fingers. Make sure to slide hand up so the curves in the brace match the mid-hand crease and the wrist.

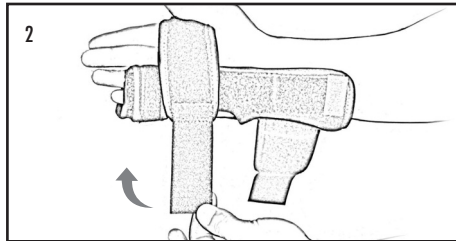

2. Secure the middle strap.

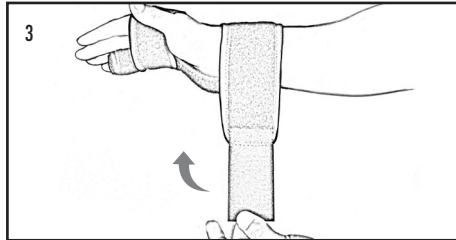


3. Secure the wrist strap.

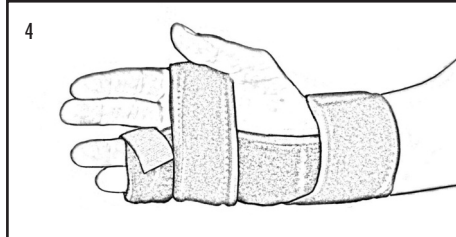


4. Finished application.


OPTIONAL:

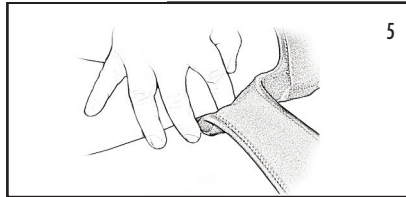

5. Optional A: If there is pressure on the ulnar styloid, you can adjust the dorsal stay. See No. 6.

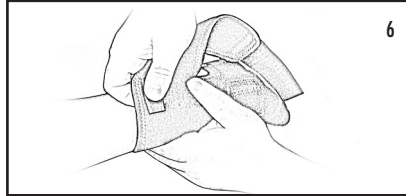

6. Optional B: Hold the dorsal stay between the index finger and thumb directly over the ulnar styloid. Bend the stay to accommodate the ulnar styloid. This will relieve the pressure.

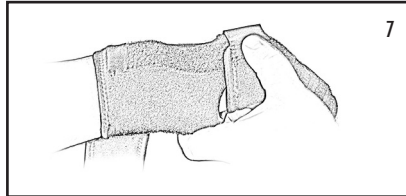

7. Finished application with extra bend for ulnar styloid pressure relief.

U.S. Patent No. 7278980, 7402149, 7455650, 7442177

HELY & WEBER

© 2010 Weber Orthopedic Inc.
DBA Hely & Weber All rights reserved.
1185 East Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com



EC REP

MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

Rev C 07/10


Radial Gauntlet Application

Warnings and Instructions: Review carefully, proper application is required


⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. This device should not be worn by persons with known allergies to neoprene.

Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.


Note: This application uses a right hand brace for a left hand injury. Use a left hand brace for a right hand injury.

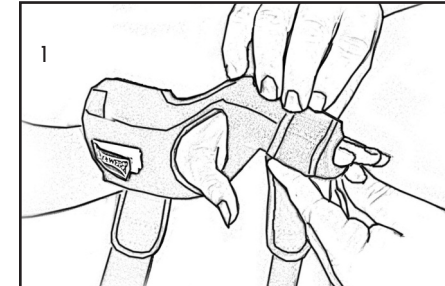


1. Apply brace, wrapping shortest strap around index and middle fingers, first.

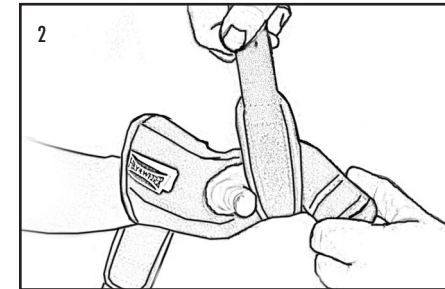

2. Secure the middle strap.

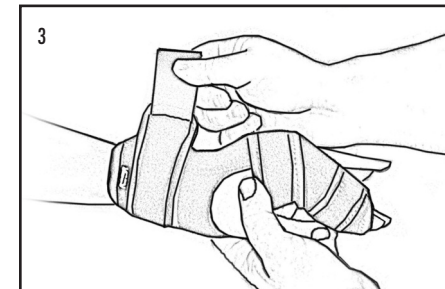

3. Secure the wrist strap.

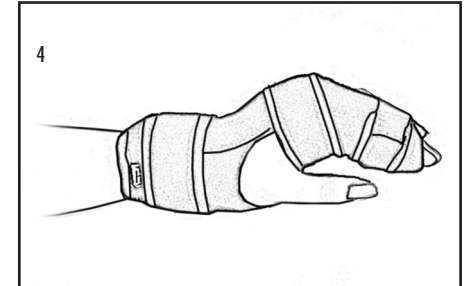

4. Finished application.



5. Finished application.

U.S. Patent No. 7278980, 7402149, 7455650, 7442177

HELY & WEBER

© 2010 Weber Orthopedic Inc.
DBA Hely & Weber All rights reserved.
1185 East Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com



EC REP

MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany


Rev C 07/10

Metacarpal Fracture Application


Warnings and Instructions: Review carefully, proper application is required

⚠ Warning: This device will not prevent or eliminate risk of injury. Do Not Over-tighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional. This device should not be worn by persons with known allergies to neoprene.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.

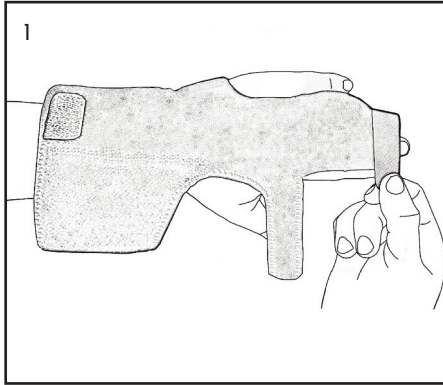


1. Apply brace by putting small finger through hole. Secure the middle and ring fingers with the distal strap.

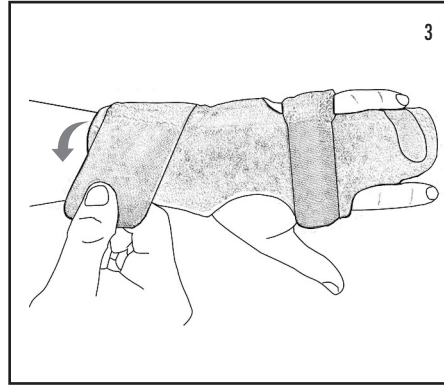

3. Secure the wrist strap.

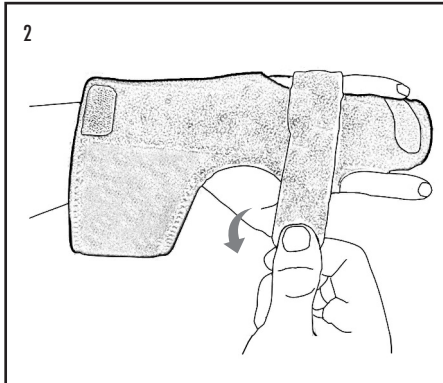

2. Secure the middle strap.

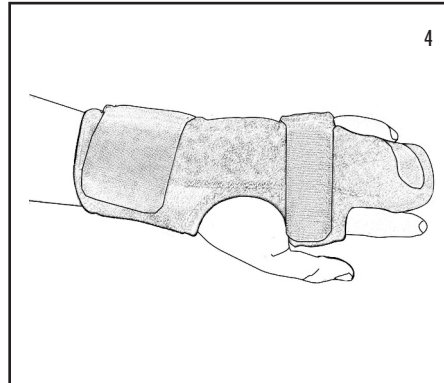

4. Finished application.

U.S. Patent No. 7278980, 7402149, 7455650, 7442177

HEL & WEBER

© 2010 Weber Orthopedic Inc.
DBA Hely & Weber All rights reserved.
1185 East Main St., Santa Paula, California 93060
California: 800-221-5465, National: 800-654-3241
International: 805-525-4244, fax: 805-933-2348
U.S. fax: 800-559-5975, www.hely-weber.com

CE

EC REP

MDSS GmbH, Schiffgraben 41
30175 Hannover, Germany

Rev C 7/10