

# Freehand CMZ Brace


**Warnings and Instructions: Review carefully, proper application is required**

**⚠ Warning:** This device will not prevent or eliminate risk of injury. Do Not Overtighten. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use immediately and consult your medical professional.


Care: Hand wash using mild soap. Rinse thoroughly. Air dry only. Do not tumble dry.




1. Place brace on hand between index finger and thumb with the short tube end lateral to the thumb.

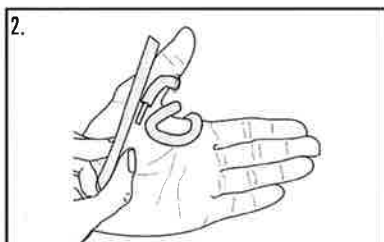


2. Pull tube around back of hand to fastener.

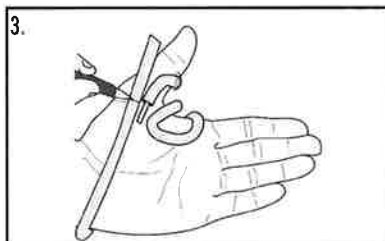



3. Mark distance to fastener.

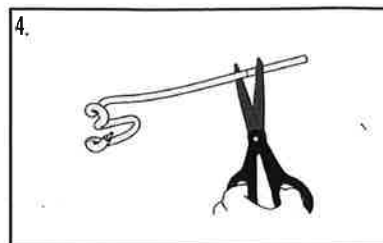


4. Remove brace from hand and cut tube at mark.

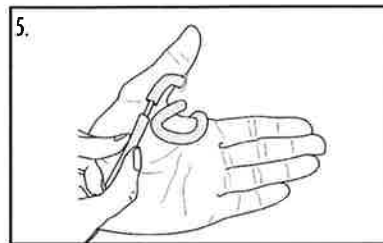

5. Place brace on hand and attach tube to fastener.

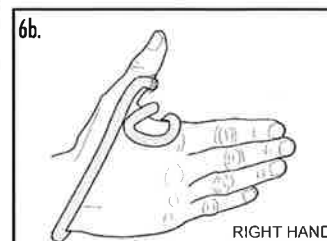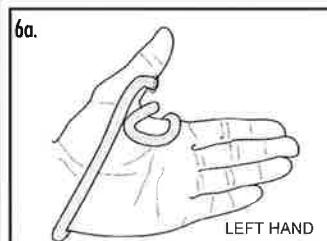

6. Finished application. When placing finished application on hand the clear thumb band should be placed on the thumb side.

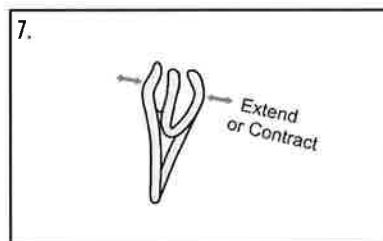

7. Adjust brace for fit and comfort.

The wire form encased in the tubing can be formed to provide a customized fit, and can be extended or contracted to accommodate individual needs and/or hand shapes and sizes.

**HELY & WEBER**

© 2014 Weber Orthopedics Inc.  
DBA: Hely & Weber all rights reserved

1185 E Main St., Santa Paula, California 93060  
California: 800-221-5465, National: 800-654-3241  
International: 805-525-4244, fax: 805-933-2348  
U.S. fax: 800-559-5975, www.hely-weber.com

**CE**  
EC REP

MDSS GmbH, Schiffgraben 41  
30175 Hannover, Germany

Rev C 1/10